PHILIPS

Manual

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What is a double-head fetoscope?

It is a low-cost 'stereo-fetoscope' (double-head) that enables simultaneously listening of fetal heart rate.

This manual supports the Heart for Two solution.

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Where, when, and for whom?

The double-head fetoscope can be used in Antenatal Care (ANC) during the 2nd half of the pregnancy (20 weeks). Midwives, midwife assistants, pregnant women and family members can use this tool in health care centers.

Why?

Objective 1

The auscultation of the fetal heart rate is an important part of monitoring fetal development, activity and well being during pregnancy and labor. Simultaneous listening to the fetal heart rate will enable the midwife to train midwife assistants in regular auscultation of the fetal heart rate. The detection of heart rate abnormalities will lead to timely medical response or referral if needed..

Objective 2

With the double-head fetoscope, the midwife can let the mother listen to her unborn child's heart rate too. This might lead to prenatal bonding, awareness and engagement towards her child and result in more frequent antenatal care visits at the health facility, as well as delivering with a skilled midwife.

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For testing purposes, two versions (A & B) of double-head fetoscope are provided.

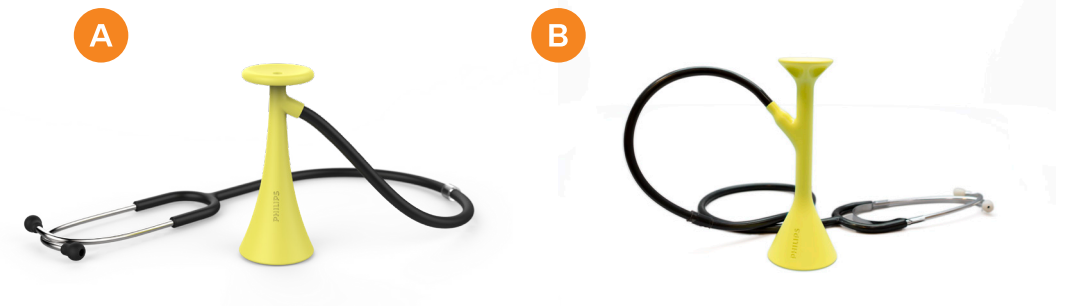
Parts

1. Hearing part for midwife, (use it as a Pinard Horn).
2. Hearing parts for other person (e.g. pregnant woman, midwives in training, etc.).

Usage

- The transmission of the fetal heart rate is best at the place, where the fetus' back is close to the mothers uterine wall.
- In order to find the best spot for listening to the fetal heart rate, the midwife needs to perform the Leopold Maneuvre (abdominal palpation) at each consultation.
- This maneuver will help her to determine the fetal lie and presentation, as well as the quantity of amniotic fluid and the fetal movement pattern. This obstetrical information is essential for monitoring the development and well being during pregnancy. This can help in predicting future risk factors as well as detecting mal presentations and obstacles to normal labor.

Please be aware that the double-head fetoscope is a prototype and should be handled with extra care.

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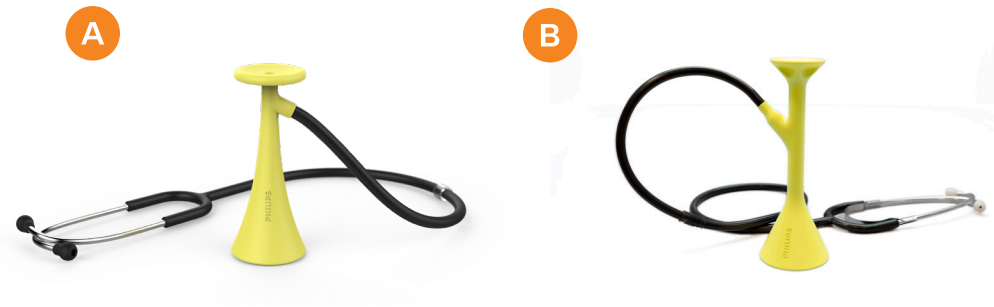
Cleaning

- Clean with a cloth and a sterilizing agent
- Do not soak

Storage

- Keep in a safe and dry place
- Keep away from sunlight
- Keep out of reach for children

Repair

- If parts should become loose they can easily be re-attached

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© 2017 Koninklijke Philips N.V. All rights reserved. Specifications are subject to change without notice.
Trademarks are the property of Koninklijke Philips N.V. (Royal Philips) or their respective owners.

www.philips.com